---

# False Lumen Embolization System

## Product Family

### Patient Implant Information

The purpose of this document is to supply you with information about your device, important safety and follow-up information.

### Device Information

#### What is the device?

The device is primarily made of a polymer which is similar in structure to a foam that has many small pores. This foam is then compressed to a small rod structure that can be pushed into blood vessels through a small tube (catheter).

#### What does the device do?

When the device is delivered through the small tube into your blood vessels, the blood moisture and body temperature cause the polymer to expand by itself to fill the area where it is implanted. Devices are inserted into the false lumen to block blood flow on purpose (embolization) when needed.

#### What is the device made of?

False Lumen Embolization System consists of two major components a permanent Implant and a Pusher system which is an external communicating device used to deploy the implant. This device is made up of various materials that can come in contact with your body, as shown in Table 1. In rare cases, some of the materials may cause an allergic reaction in some patients. If you have any questions or concerns, contact your healthcare provider.

**Table 1: False Lumen Embolization System Materials**

Device Name	Model Number	Permanent Implant Component	Materials
False Lumen Embolization System	<ul style="list-style-type: none"> <li>• FLE-18x1</li> <li>• FLE-18x3</li> <li>• FLE-18x5</li> <li>• FLE-24x1</li> <li>• FLE-24x3</li> <li>• FLE-24x5</li> <li>• FLE-36x1</li> <li>• FLE-36x3</li> <li>• FLE-36x5</li> </ul>	Radiopaque markerband	Platinum-Iridium alloy
		Polymer	Polyurethane
		Tether wire	Platinum-Iridium alloy
		Detachment Collar	Platinum-Iridium alloy
		Adhesive	Acrylic
		<b>Pusher Component</b>	<b>Materials</b>
		Proximal Hub	Polycarbonate/ABS
		Shaft	PEBAX
		Transfer Tube	High Density polyethylene
		Inner Actuator	Stainless Steel
		Adhesive	Ethyl cyanoacrylate

Please note that the model number of your device can be located on your Implant Card.

### What is the Intended Use of the device?

The False Lumen Embolization System is indicated for use in patients with subacute or chronic aortic dissection. After or during the primary entry tear repair procedure, the False Lumen Embolization System is intended to obstruct or reduce the rate of blood flow in the false lumen.

The False Lumen Embolization System is intended for use by physicians trained and experienced in endovascular management of aortic dissection, embolization techniques, angiographic techniques, and interventional procedures. Standard techniques for placement of vascular access sheaths, introducer sheaths, and guidewires should be employed.

### Contraindications

The False Lumen Embolization System is contraindicated in patients with known sensitivities or allergies to stainless steel, platinum, polyurethane, or solder (tin, silver).

### Possible Side effects

Possible side effects associated with either the device or the implantation procedure that may occur and/or require intervention include, but are not limited to:

- Access, delivery, and deployment events (e.g. access failure, deployment difficulties/failures, failure to deliver the device, misplacement, and insertion or removal difficulty)
- Adjunctive device damage, thrombosis, or occlusion

- Allergic reaction and/or anaphylactoid response (e.g., to x-ray contrast dye, antiplatelet therapy, device materials), hypersensitivity
- Amputation
- Anesthetic complications and subsequent attendant problems (e.g., aspiration)
- Angina
- Aortic branch vessel occlusion or obstruction
- Aortic expansion (e.g., aneurysm, false lumen, landing zone, lesion) including new dissection or aneurysm formation and effect on abdominal or thoracic diameter
- Aortic valve damage
- Aortic vessel or aneurysm rupture
- Arterial or venous thrombosis and/or pseudoaneurysm
- Atelectasis / pneumonia
- Bleeding, hemorrhage, hematoma, hemolysis, or coagulopathy
- Blindness
- Blood loss
- Bowel complications (e.g., ileus, adynamic ileus, transient ischemia, infarction, necrosis)
- Cardiac complications and subsequent attendant problems (e.g., arrhythmia, tamponade, myocardial infarction, congestive heart failure, hypotension, hypertension)
- Change in mental status
- Claudication (e.g., buttock, lower limb)
- Contrast toxicity
- Death
- Device damage, fracture, fragmentation, failure to expand, partially or totally
- Device explantation, partial or total
- Device migration
- Dissection, perforation, or rupture of the aortic vessel & surrounding vasculature
- Edema
- Embolization (unintended, micro and macro) with transient or permanent ischemia or infarction, including air embolism
- Endoleak (entry flow)
- Erectile dysfunction, impotence
- Excessive or inappropriate radiation exposure
- Extrusion/erosion
- Fever and localized inflammation
- Fistula (e.g., aortoenteric, arteriovenous, aortoesophageal, aortobronchial)
- Gastrointestinal bleeding/complications
- Genitourinary complications and subsequent attendant problems (e.g., ischemia, erosion, fistula, urinary incontinence, hematuria, infection)
- Hepatic failure
- Infarction
- Infection of the aneurysm, device or access site, including abscess formation, transient fever, and pain
- Lymphatic complications and subsequent attendant problems (e.g., lymph fistula, lymphocele)
- Nerve injury
- Neurologic complications (local or systemic) and subsequent attendant problems (e.g., stroke, transient ischemic attack, paraplegia, paraparesis, spinal cord shock, paralysis)
- Neuropathy, femoral neuropathy
- Occlusion, stenosis (including restenosis), or thrombosis – venous or arterial

- Pain
- Paresthesia
- Peripheral malperfusion or ischemia and subsequent attendant problems (e.g., spinal cord ischemia, visceral vessel ischemia, limb ischemia)
- Persistent perfusion of the false lumen (and/or incomplete thrombosis)
- Post implantation syndrome
- Pulmonary embolism
- Pulmonary/respiratory complications and subsequent attendant problems (e.g., pneumonia, respiratory failure, prolonged intubation)
- Renal complications and subsequent attendant problems (e.g., artery occlusion, contrast toxicity, insufficiency, failure)
- Reoperation
- Sepsis
- Seroma
- Shock
- Surgical conversion
- Tissue necrosis
- Vascular access site complications, including infection, pain, hematoma, pseudoaneurysm, arteriovenous fistula
- Vascular spasm or vascular trauma or damage (e.g., iliofemoral vessel dissection, bleeding, rupture)
- Wound complications and subsequent problems (e.g., dehiscence, infection)

Should a serious device-related event occur, immediately notify Shape Memory Medical of the incident and the applicable competent authority or regulatory body.

### **After your Procedure**

Please carry your Implant Card(s) with you. Whenever you visit any healthcare facility, refer to the Implant Card(s), particularly for any additional diagnostic procedures (e.g., MRI). After your procedure, always follow your doctor's recommendations to ensure continuing safe and effective results.

After your implant surgery, your healthcare provider will give you instructions about the follow up care. Check with your healthcare provider for details regarding at-home care, medications, diet and physical activity. Follow the instructions provided by the healthcare provider to ensure the best possible results.

### **Follow-up**

Your doctor will evaluate you and recommend adequate follow-up based on your needs. Follow-up should be maintained even in the absence of clinical symptoms. Complications may not always show symptoms but can be seen on imaging at your follow-up appointments.

If you continue to experience symptoms or start to feel unwell and/or have any concerns, speak with your doctor without further delay.

## How Long Will I Have the Device?

Your implant is a permanent implant and is not intended to be removed once implanted. Talk to your healthcare provider about the risks and benefits of this device, and any necessary follow-up procedures. Consult with your healthcare provider if you have any questions about the lifetime of your implant device.

## MRI Safety Information

MR Conditional

Important: Before getting an MRI scan, tell your healthcare provider that you have an implanted embolization plug. Show your Implant Card to your healthcare provider.

An MRI scan is a type of medical imaging that uses magnetic fields to create an internal view of the body. Your device is rated as MR Conditional, meaning that your healthcare provider must follow the MRI safety instructions below.

Non-clinical testing and MRI simulations were performed to evaluate the Implant in its worst-case (largest) configuration. Non-clinical testing demonstrated that the Implant is MR Conditional. A patient implanted with one or more Implants may be scanned safely in an MR system under the following conditions:

- Static, horizontal, magnetic field of 1.5-Tesla or 3-Tesla, only
- Maximum spatial gradient magnetic field of 10,000-gauss/cm (100-T/m)
- Maximum whole body averaged (WBA) specific absorption rate (SAR) in Normal Operating Mode
  - 1.5-Tesla/64-MHz, WBA SAR, 1-W/kg
  - 3-Tesla/128MHz, WBA SAR, 2-W/kg
- Scan Duration and Wait Time
  - 1.5-Tesla/64-MHz, WBA SAR, 1-W/kg: 10 minutes continuous RF exposure, followed by 20 minute wait period, repeated twice in 60 minutes. Alternatively, the patient may undergo 15 minutes continuous RF exposure, followed by a 45 minute wait period (once per 60 minutes).
  - 3-Tesla/128-MHz, WBA SAR, 2-W/kg: 60 minutes continuous RF exposure (i.e., per pulse sequence or back-to-back sequences/series without breaks).

Under the scan conditions defined, the Implant is expected to produce a temperature rise of less than or equal to 6°C.

In non-clinical testing, the image artifact caused by the Implant extends approximately 4 mm from the device when imaged using a gradient echo pulse sequence and a 3-Tesla MR system.

### Contact Information

If you have any questions that are not addressed in this document, ask your healthcare provider. To contact Shape Memory Medical, use the contact information at the end of this document.

### Explanation of symbols on the implant card

	Patient Name or Patient ID
	Date of Implantation
	Name and address of the Implanting Healthcare institution/provider
	Name and Address of the Manufacturer
	Website Information for patients
	Model Number
	Device Name
	Lot Number
	UDI Number
	MR Conditional

**Shape Memory Medical, Inc.**  
 1321 Ridder Park Dr., Suite 10  
 San Jose, CA 95131, USA  
[www.shapemem.com](http://www.shapemem.com)  
 Phone: +1 408 649 5175

**MedR-AR Services B.V.R**  
 Kloosterweg 1  
 NL 6412 CN Heerlan  
 Phone: +31 45 303 0006

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